

Horse DNA Submission Form

Instructions:

1. Thoroughly wash and dry your hands.
2. Pull (DO NOT CUT) 20-30 tail or mane hairs by wrapping hairs around forefinger. For foals, use tail hairs only. Grasp hair close to the animal's body to insure roots are included. Pull straight toward your body. Visually check that roots are attached to hair. DO NOT TOUCH ROOT BULBS.
3. Place sample in shaded box (below) with roots at left. Tape in place with tape over hair shaft on right.
4. Fold submission form backwards so that logo and identification are on the outside and hair is covered. See fold lines for details. Do not seal submission form with tape or staple shut.
5. Place in envelope and mail to the address below.
6. If mailing from outside the United States, a current import permit must accompany the sample.


Mail To:

LifeLine Labs, LLC- Genetics Division
P.O. Box 911014
Lexington, KY 40591

For FedEx, UPS, DHL, etc

LifeLine Labs, LLC - Genetics Division
1088c Wellington Way
Lexington, KY 40513

Second fold (Fold back to cover hair. Do not cut, tape or staple)

<p>Owner Name: Address:</p>		Fold with LOGO on outside
<p>Animal being tested Name: Registration: Barn Name: Color: Date of Birth: Sex: Breed: Thoroughbred</p>	<p>Parent Information SIRE Name: Registration: DAM Name: Registration:</p>	

First fold (Fold back. Do not cut, tape or staple)

Place Roots Here	Tape Here	Place rest of hair here. Coil if necessary.
---------------------------------	----------------------	--

Pull 20-30 tail or mane hairs of animal being tested and place above. Only one animal per form.